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Catalogue of instruments for vitreoretinal surgery

Microsurgical Instruments Co., Ltd.
Kazan Medical Instruments Plant JSC

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CONTENT

Any kinds of handles
Solid instrument
Various kinds of tips
25 Gauge instruments
23 Gauge instruments
20 Gauge instruments
Foreign body removal forceps
Miscellaneous instruments
Membrane instruments
Set for seamless surgery
Instruction for operation, cleaning and sterilization
Guarantee obligations

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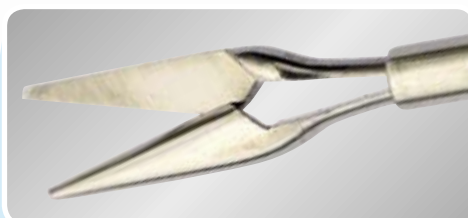
Handle for vitreoretinal instrument,
squeeze model, two fingers control,
for tips with thread,
titanium

Dear Ladies and Gentlemen,

«Microsurgical Instruments» Co. Ltd., Kazan, presents herewith their Catalogue of instruments for vitreo retinal surgery.

The analytical approach to the doctors' ideas, to new technological processes of manufacturing the instruments and to their design as well as the application of titanium alloys enable us to be continuously among the world leaders in the fabrication of microsurgical instruments.

The high professional level of engineers and qualified workers as well as the use of the precision equipment ensure the stable quality and the possibility to realize our products in the world market. Components of leading companies from the USA, Germany and Japan are used for manufacturing our instruments for vitreo retinal surgery.

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Our Quality Management System complies with ISO 9001 and ISO 13485 International standards.

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Handle for vitreoretinal instrument,
squeeze model, one finger control,
for tips with thread,
titanium

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Universal handle, adjustable,
for vitreoretinal instrument,
for tips with thread,
with aspiration channel, titanium

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Any instrument may be manufactured in the form
of a solid non-detachable unit. Therefore letter "S" (solid)
must be added at the end of the code of the instrument. Example: V-021S.

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**Instruments with straight
long tube**

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**Instruments with straight
short tube**

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**Instruments with curved
short tube**

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
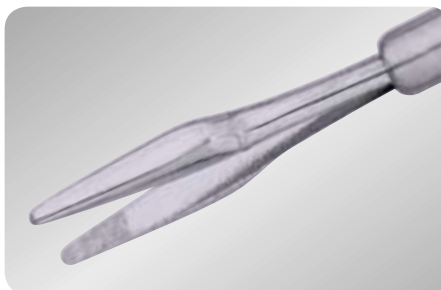
Forceps, straight, smooth, 1.8 mm long branches, TC coated tips, tube diameter 0.5 mm, 30 mm long tube

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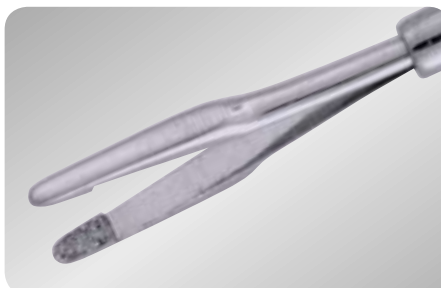Forceps, straight, with platform, 1.8 mm long branches, TC coated tips, tube diameter 0.5 mm, 30 mm long tube

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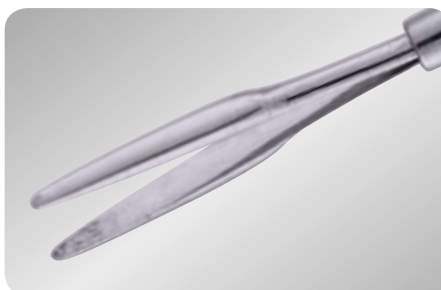
Forceps, straight, with serrations, 1.8 mm long branches, tube diameter 0.5 mm, 30 mm long tube

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
Forceps, straight, smooth, elongated, 2.4 mm long branches, TC coated tips, tube diameter 0.5 mm, 30 mm long tube

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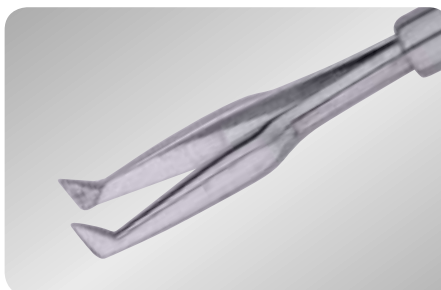
Forceps, straight, with grasping tips, 1.8 mm long branches, TC coated tips, tube diameter 0.5 mm, 30 mm long tube

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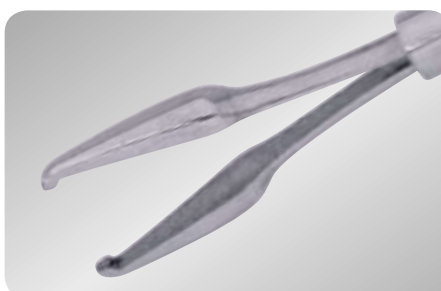
Forceps, straight, with platform, 1.3 mm long branches, TC coated tips, tube diameter 0.5 mm, 32 mm long tube

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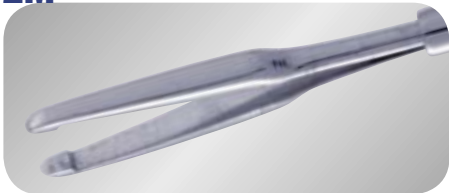
Forceps, with nail-shaped grasping
tips, elongated,
2.5 mm long branches,
TC coated tips,
tube diameter 0.5 mm,
30 mm long tube

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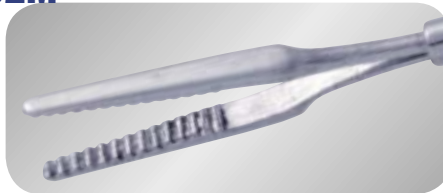
Forceps, with serrations, elongated,
3.2 mm long branches,
tube diameter 0.5 mm,
30 mm long tube

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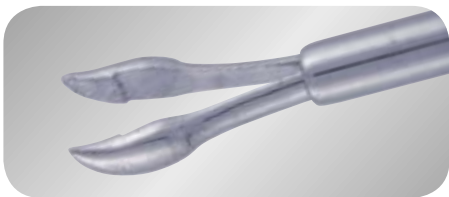
Forceps, asymmetric, with platform,
1.3 mm long branches,
TC coated tips,
tube diameter 0.5 mm,
30 mm long tube

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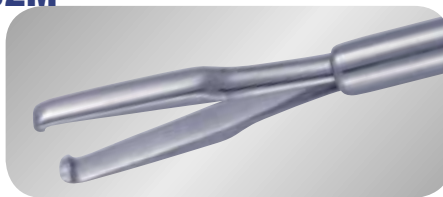
Forceps,
with grasping nail-shaped tips,
2.3 mm long branches,
TC coated tips,
tube diameter 0.5 mm,
30 mm long tube

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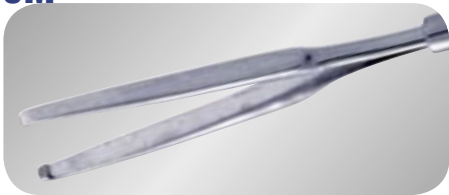ECKARDT forceps,
with nail-shaped grasping tips,
elongated, 4.0 mm long branches,
TC coated tips, tube diameter 0.5 mm,
30 mm long tube

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Scissors, straight,
2.0 mm long cutting edges,
tube diameter 0.5 mm,
30 mm long tube

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Scissors, vertical, cutting edge of
fixed branch angled at 45°,
tube diameter 0.5 mm,
30 mm long tube

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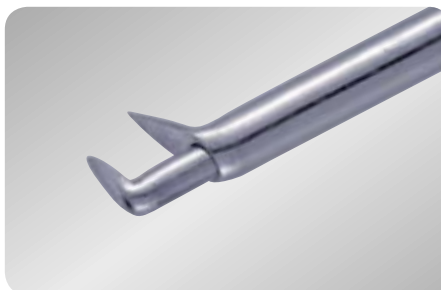
Forceps for anterior capsule of nucleus, 1.3 mm long branches, TC coated tips, tube diameter 0.65 mm, radius of the tube curve 42 mm, 18 mm long tube

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Forceps for capsulorhexis, end-faced, for aspiration, length of fixed branch 1.65 mm, tube diameter 0.65 mm, radius of the tube curve 35 mm, 18 mm long tube

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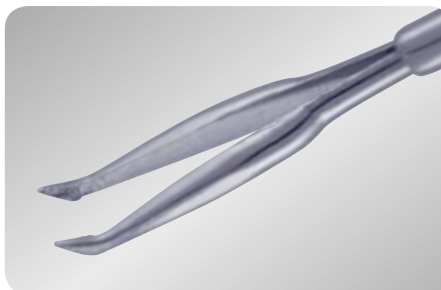
Forceps for capsulorhexis, 1.8 mm long branches, TC coated tips, tube diameter 0.65 mm, radius of the tube curve 30 mm, 20 mm long tube

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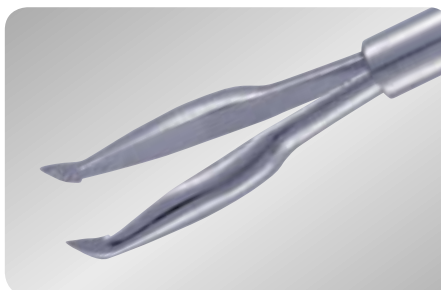
Forceps for capsulorhexis, medium size, 2.5 mm long branches, TC coated tips, tube diameter 0.65 mm, 30 mm long tube

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Forceps for capsulorhexis, curved, with platform, 1.8 mm long branches, branches angled at 45°, TC coated tips, tube diameter 0.65 mm, radius of the tube curve 30 mm, 25 mm long tube

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
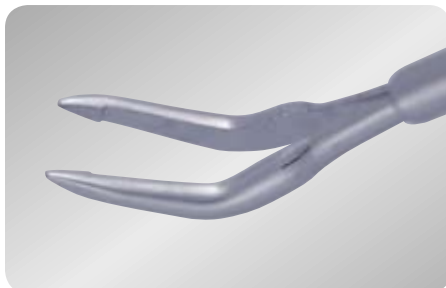
Forceps for capsulorhexis, curved, with platform, branches angled at 45°, TC coated tips, tube diameter 0.65 mm, radius of the tube curve 32 mm, 23 mm long tube

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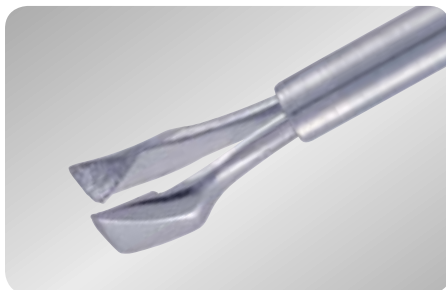IKEDA forceps for capsulorhexis, 1.2 mm long branches, TC coated tips, tube diameter 0.65 mm, radius of the tube curve 30 mm, 20 mm long tube

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Forceps, straight, smooth, 1.8 mm long branches, TC coated tips, tube diameter 0.65 mm, 30 mm long tube

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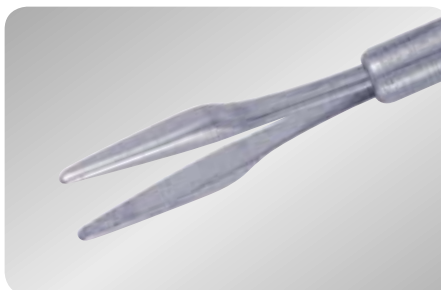
Forceps, straight, with serrations, 1.8 mm long branches, tube diameter 0.65 mm, 30 mm long tube

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
Forceps, straight, smooth,
elongated, 2.4 mm long branches,
TC coated tips,
tube diameter 0.65 mm,
30 mm long tube

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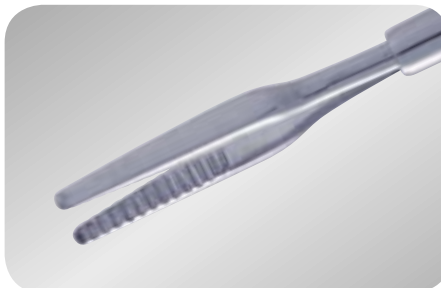Forceps, straight, with serrations,
elongated, 3.2 mm long branches,
tube diameter 0.65 mm,
30 mm long tube

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Forceps, with serrations,
2.2 mm long branches,
tube diameter 0.65 mm,
radius of the tube curve 30 mm,
30 mm long tube

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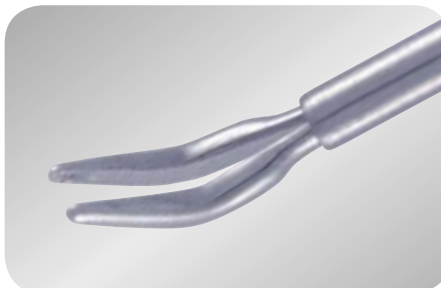
Forceps, curved, with serrations,
1.9 mm long branches,
tube diameter 0.65 mm,
30 mm long tube

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
Forceps, curved, elongated,
2.2 mm long branches,
TC coated tips,
tube diameter 0.65 mm,
30 mm long tube

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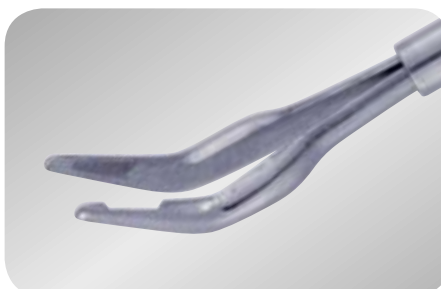Forceps, with slot,
branches angled at 45°,
2.2 mm long branches,
tube diameter 0.65 mm,
30 mm long tube

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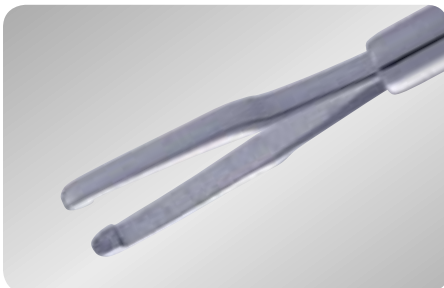
Forceps, straight, with platform, 1.8 mm long branches, TC coated tips, tube diameter 0.65 mm, 30 mm long tube

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
Forceps, straight, with platform, TC coated tips, tube diameter 0.65 mm, 30 mm long tube

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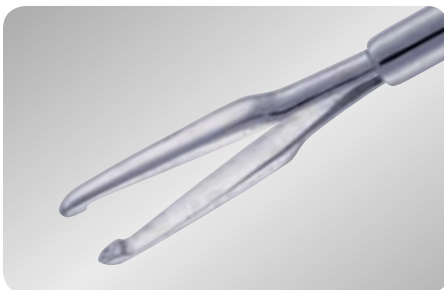
Forceps, with nail-shaped grasping tips, elongated, 2.5 mm long branches, TC coated tips, tube diameter 0.65 mm, 30 mm long tube

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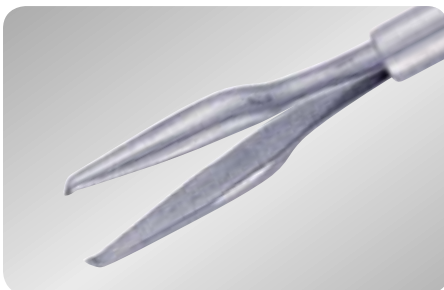
Forceps, with displaced micro teeth, 2.5 mm long branches, tube diameter 0.65 mm, 30 mm long tube

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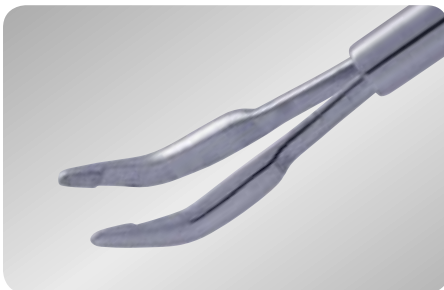
Forceps, curved, with platform, 1.9 mm long branches, branches angled at 45°, TC coated tips, tube diameter 0.65 mm, 30 mm long tube

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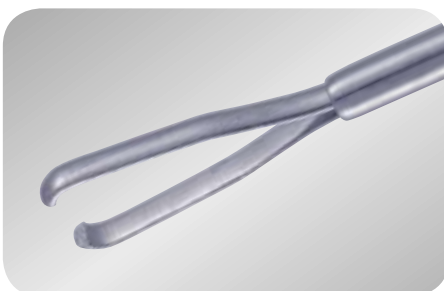
Forceps for ILM, 2.5 mm long branches, TC coated tips, tube diameter 0.65 mm, 30 mm long tube

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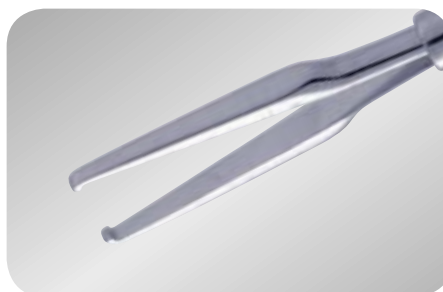
ECKARDT forceps,
with nail-shaped grasping tips,
2.5 mm long branches,
TC coated tips,
tube diameter 0.65 mm,
30 mm long tube

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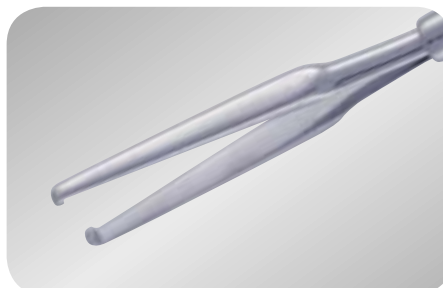
ECKARDT forceps,
with nail-shaped grasping tips,
elongated, 4.0 mm long branches,
TC coated tips,
tube diameter 0.65 mm,
30 mm long tube

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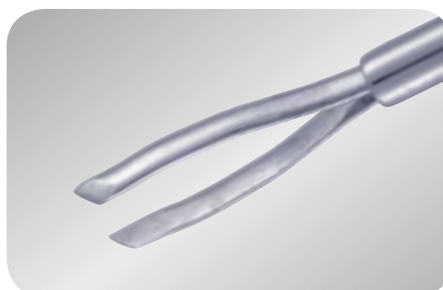Forceps, with grasping tips,
2.5 mm long branches,
tube diameter 0.65 mm,
30 mm long tube

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Scissors, horizontal, with branches
curved to the left, radius of the curve
of branches 2.5 mm,
branches angled to the axis of the
tube at 30°, tube diameter 0.65 mm,
18 mm long tube

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Scissors, horizontal, with branches
curved to the right, radius of
the curve of branches 2.5 mm,
branches angled to the axis
of the tube at 30°,
tube diameter 0.65 mm,
18 mm long tube

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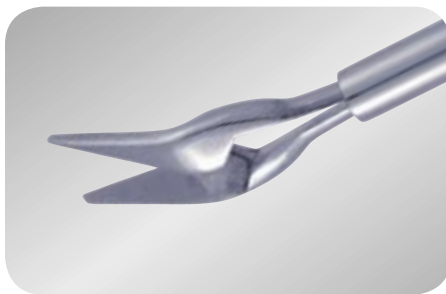
Scissors, horizontal,
branches angled at 30°,
cutting edges 1.3 mm long,
tube diameter 0.65 mm,
18 mm long tube

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
Scissors, angled at 30° along the
plane of the blades,
tube diameter 0.65 mm,
30 mm long tube

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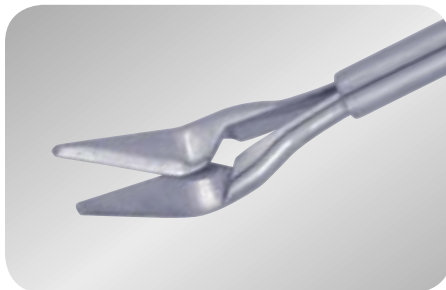
Scissors, angled at 45° along the
plane of the blades,
tube diameter 0.65 mm,
30 mm long tube

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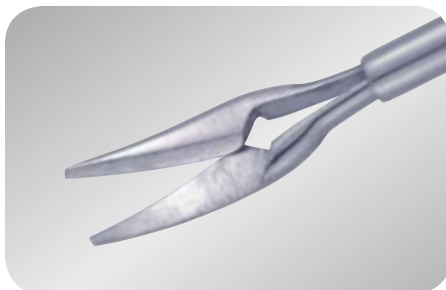
Scissors, curved along the plane
of the blades,
radius of the curve 3.0 mm,
tube diameter 0.65 mm,
30 mm long tube

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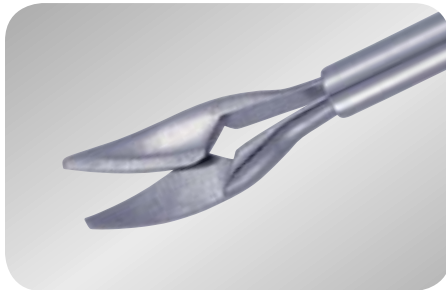
Scissors, blunt-pointed, shortened,
curved along the plane of the
blades, radius of the curve 2.0 mm,
length of cutting edge 1.2 mm,
tube diameter 0.65 mm,
30 mm long tube

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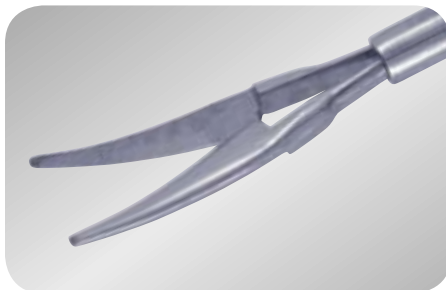
Scissors, curved along the plane
of the blades,
radius of the curve 12.0 mm,
tube diameter 0.65 mm,
30 mm long tube

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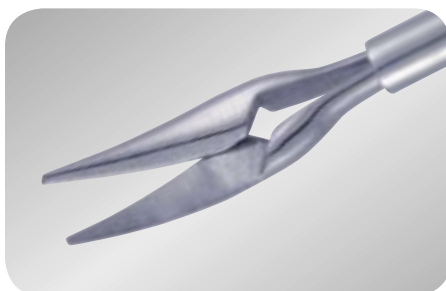
Scissors, subretinal, curved along
the plane of the blades,
radius of the curve 7.6 mm,
tube diameter 0.65 mm,
30 mm long tube

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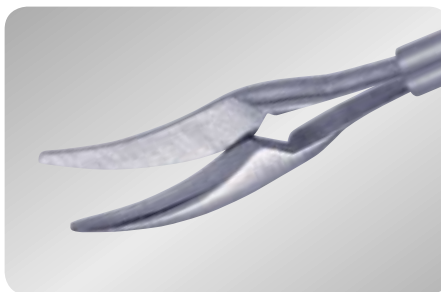
Scissors, curved,
blunt-pointed, sickle-shaped,
radius of the curve 5.0 mm,
length of cutting edge 2.35 mm,
tube diameter 0.65 mm, 30 mm
long tube

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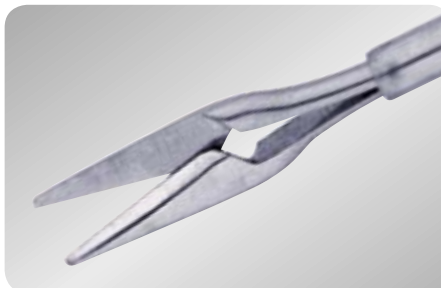
Scissors, straight,
length of cutting edge 2.2 mm,
tube diameter 0.65 mm,
15 mm long tube

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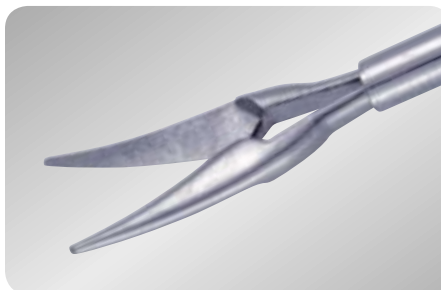
Forceps for capsulotomy,
bent to the right,
2.5 mm long branches,
radius of the curve 12 mm,
tube diameter 0.65 mm,
30 mm long tube

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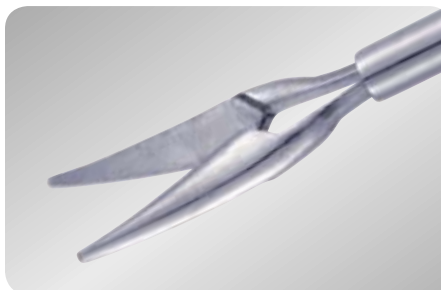
Forceps for capsulotomy,
bent to the left,
2.5 mm long branches,
radius of the curve 12 mm,
tube diameter 0.65 mm,
30 mm long tube

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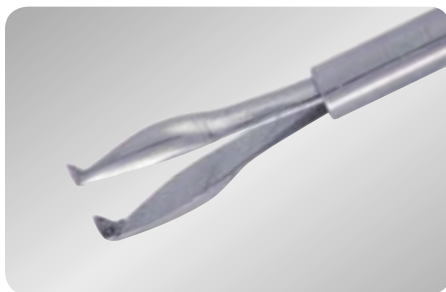Forceps for capsulorhexis, 1.8 mm long branches, TC coated tips, tube diameter 0.9 mm, 30 mm long tube

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Forceps, straight, smooth, 1.8 mm long branches, TC coated tips, tube diameter 0.9 mm, 30 mm long tube

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Forceps, straight, with serrations, 1.8 mm long branches, tube diameter 0.9 mm, 30 mm long tube

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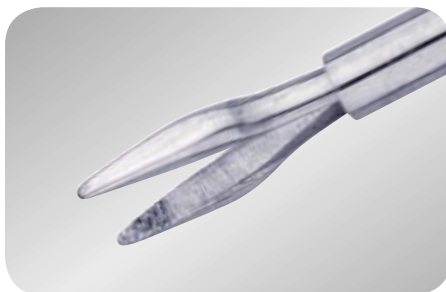
Forceps straight, with serrations, platform and groove, 2.6 mm long branches, TC coated tips, tube diameter 0.9 mm, 30 mm long tube

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Forceps, straight, smooth, elongated, 2.4 mm long branches, TC coated tips, tube diameter 0.9 mm, 30 mm long tube

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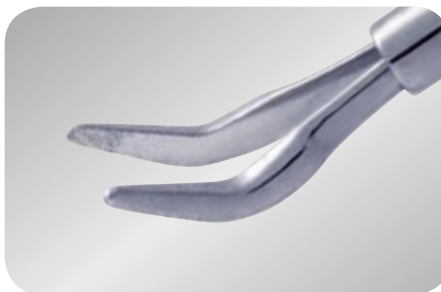
Forceps, curved, with serrations, 1.9 mm long branches, tube diameter 0.9 mm, 30 mm long tube

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
Forceps, curved, elongated, 1.9 mm long branches, TC coated tips, tube diameter 0.9 mm, 30 mm long tube

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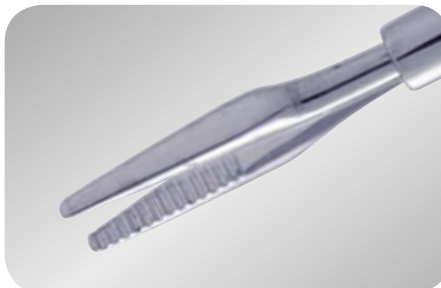Forceps, straight, with serrations, elongated, 3.2 mm long branches, tube diameter 0.9 mm, 30 mm long tube

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Forceps, straight, with serrations and platform, elongated, 2.2 mm long branches, TC coated platform, tube diameter 0.9 mm, 30 mm long tube

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Forceps, straight, with platform, 1.8 mm long branches, TC coated tips, tube diameter 0.9 mm, 30 mm long tube

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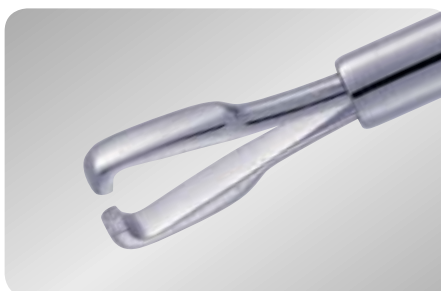
Forceps, curved, with platform, 1.9 mm long branches, branches angled at 45°, TC coated tips, tube diameter 0.9 mm, 30 mm long tube

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
Forceps, with nail-shaped grasping tip, 2.0 mm long branches, TC coated tips, tube diameter 0.9 mm, 30 mm long tube

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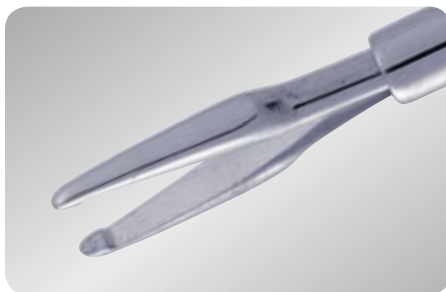
Forceps, with nail-shaped grasping tip, elongated, 2.5 mm long branches, TC coated tips, tube diameter 0.9 mm, 30 mm long tube

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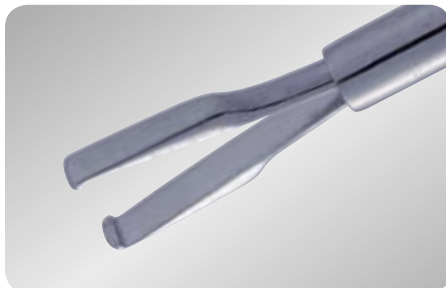
Forceps, with nail-shaped grasping tip, 2.3 mm long branches, TC coated tips, tube diameter 0.9 mm, 30 mm long tube

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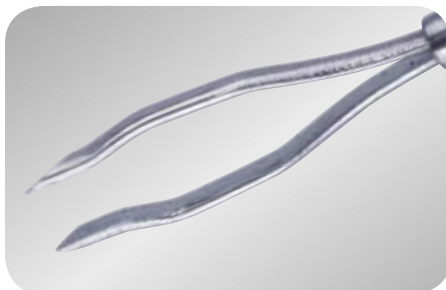
Forceps, with grasping tips, 2.5 mm long branches, tube diameter 0.9 mm, 30 mm long tube

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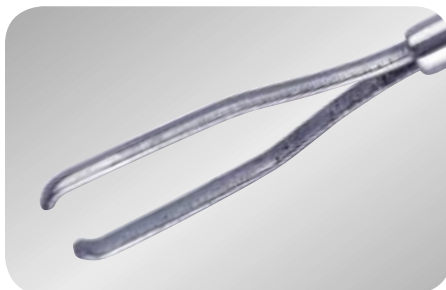Forceps for ILM, 2.5 mm long branches, TC coated tips, tube diameter 0.9 mm, 30 mm long tube

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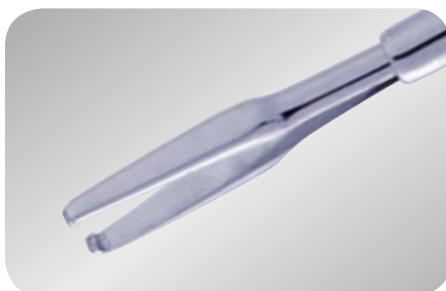
ECKARDT forceps, with nail-shaped grasping tips, 2.5 mm long branches, TC coated tips, tube diameter 0.9 mm, 30 mm long tube

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
ECKARDT forceps, with nail-shaped grasping tips, elongated, 4.0 mm long branches, TC coated tips, tube diameter 0.9 mm, 30 mm long tube

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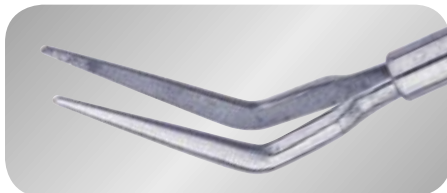
Forceps, subretinal, curved, branches angled at 60°, TC coated tips, tube diameter 0.9 mm, 30 mm long tube

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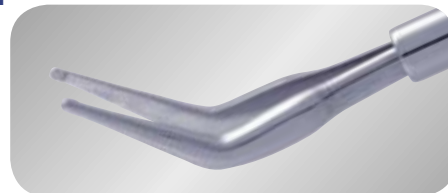
Forceps, subretinal, curved, with platform, branches angled at 45°, TC coated tips, tube diameter 0.9 mm, 30 mm long tube

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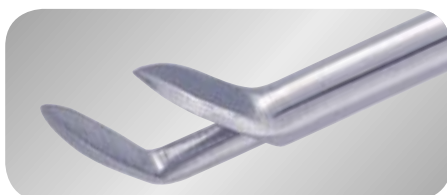Forceps, separating, with sharp-pointed tips, branches 2.8 mm high, branches angled at 60°, tube diameter 0.9 mm, 30 mm long tube

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Forceps, separating, with blunt-pointed tips, branches 2.8 mm high, branches angled at 60°, tube diameter 0.9 mm, 30 mm long tube

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Forceps, separating, with sharp-pointed tips, branches 2.8 mm high, branches angled at 45°, tube diameter 0.9 mm, 30 mm long tube

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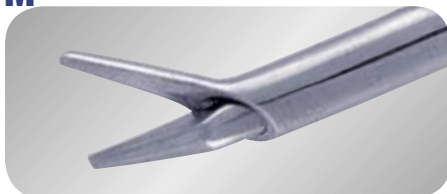
Forceps, separating, with blunt-pointed tips, branches 2.8 mm high, branches angled at 45°, tube diameter 0.9 mm, 30 mm long tube

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Scissors, horizontal, sharp-pointed, branches angled at 45°, cutting edges 1.8 mm long, tube diameter 0.9 mm, 30 mm long tube

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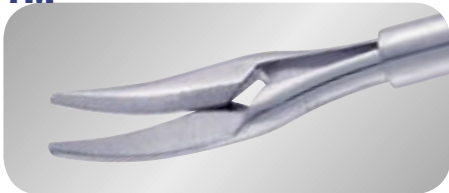
Scissors, horizontal, with branches curved to the right, radius of the curve 2.0 mm, branches angled to the axis of the tube at 40°, tube diameter 0.9 mm, 30 mm long tube

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
Scissors, elongated, curved along
the plane of the blades,
radius of the curve 4.0 mm,
tube diameter 0.9 mm,
30 mm long tube

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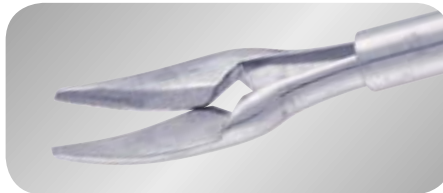
Scissors, elongated, curved along
the plane of the blades,
radius of the curve 12.0 mm,
tube diameter 0.9 mm,
30 mm long tube

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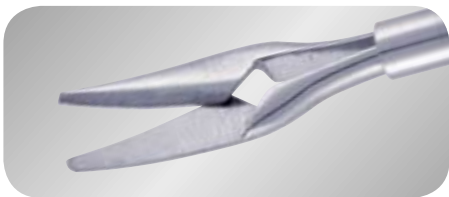Scissors, subretinal, curved along
the plane of the blades,
radius of the curve 7.6 mm,
tube diameter 0.9 mm,
30 mm long tube

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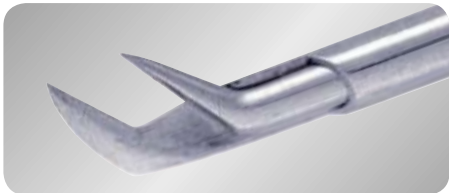
Scissors, vertical, with slightly
blunted tips of branches,
fixed branch angled at 45°,
1.5 mm long blades,
tube diameter 0.9 mm,
30 mm long tube

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Scissors, vertical,
with sharp-pointed tips of branches,
fixed branch angled at 45°,
1.5 mm long blades,
tube diameter 0.9 mm,
30 mm long tube

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Scissors, vertical, with olive on the
tip of fixed branch,
fixed branch angled at 45°,
length of blade with olive 1.75 mm,
tube diameter 0.9 mm,
30 mm long tube

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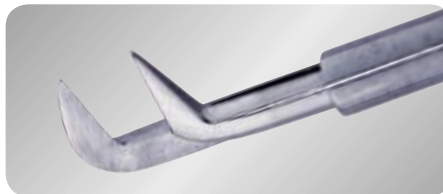
Scissors, vertical, with slightly
blunted tips of branches,
fixed branch angled at 90°,
1.5 mm long blades,
tube diameter 0.9 mm,
30 mm long tube

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Scissors, vertical,
with sharp-pointed tips of branches,
fixed branch angled at 90°,
1.5 mm long blades,
tube diameter 0.9 mm,
30 mm long tube

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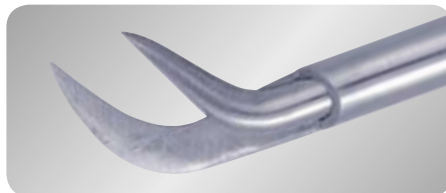
Scissors, curved, blunt-pointed, length of cutting edge 2.5 mm, sickle-shaped curve of blades, radius of the curve 5.0 mm, tube diameter 0.9 mm, 30 mm long tube

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LAMBERT scissors, vertical, 2.1 mm long branches, tube diameter 0.9 mm, 30 mm long tube

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Scissors, curved, sickle-shaped, bent to the right, radius of the curve 5.0 mm, length of cutting edge 2.7 mm, tube diameter 0.9 mm, 30 mm long tube

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Scissors, curved, sickle-shaped, bent to the right, radius of the curve 5.0 mm, length of cutting edge 2.0 mm, tube diameter 0.9 mm, 30 mm long tube

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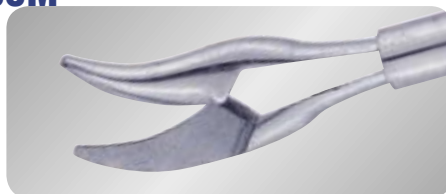
Scissors, curved, sickle-shaped, bent to the left, radius of the curve 5.0 mm, length of cutting edge 2.7 mm, tube diameter 0.9 mm, 30 mm long tube

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Scissors, curved, sickle-shaped, bent to the left, radius of the curve 5.0 mm, length of cutting edge 2.0 mm, tube diameter 0.9 mm, 30 mm long tube

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Scissors, straight, length of cutting edge 3.0 mm, tube diameter 0.9 mm, 30 mm long tube

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Scissors, straight, length of cutting edge 1.85 mm, tube diameter 0.9 mm, 30 mm long tube

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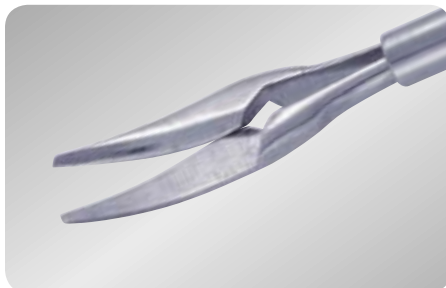
Scissors for capsulotomy, bent to the right, 2.6 mm long branches, radius of the curve 12 mm, tube diameter 0.9 mm, 30 mm long tube

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
Scissors for capsulotomy, bent to the left, 2.6 mm long branches, radius of the curve 12 mm, tube diameter 0.9 mm, 30 mm long tube

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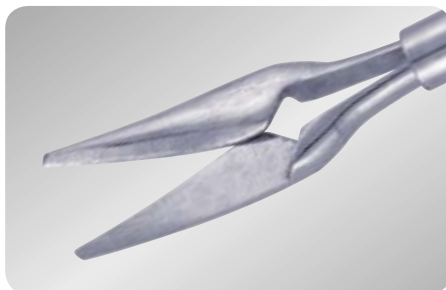
Scissors for capsulotomy, bent to the left, 3.6 mm long branches, radius of the curve 12 mm, tube diameter 0.9 mm, 30 mm long tube

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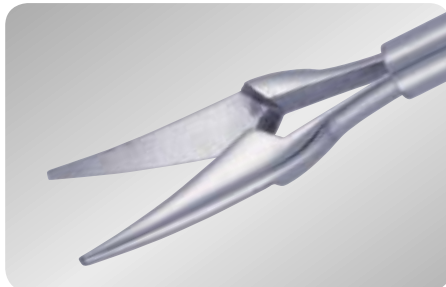
Scissors for capsulotomy, bent to the right, 3.6 mm long branches, radius of the curve 12 mm, tube diameter 0.9 mm, 30 mm long tube

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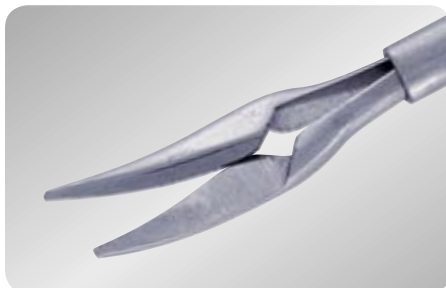Scissors for capsulotomy, bent to the left, 2.6 mm long branches, radius of the curve of branches 12 mm, tube diameter 0.9 mm, radius of the curve of the tube 30 mm, 18 mm long tube

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Scissors for capsulotomy, bent to the right, 2.6 mm long branches, radius of the curve of branches 12 mm, tube diameter 0.9 mm, radius of the curve of the tube 30 mm, 18 mm long tube

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Forceps, with two arms, open in normal position, 3.0 mm long branches, tube diameter 1.2 mm, 30 mm long tube

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Forceps, with three arms, open in normal position, 0.55 mm wide branches, tube diameter 1.2 mm, 30 mm long tube

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Forceps, with cup-shaped jaws, open in normal position, tube diameter 1.2 mm, 30 mm long tube

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Forceps, with fenestrated jaws, 2.5 mm long branches, TC coated tips, tube diameter 1.2 mm, 30 mm long tube

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Forceps, with two arms, open in normal position, tube diameter 0.9 mm, 30 mm long tube

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
Forceps, with three arms, open in normal position, 0.35 mm wide branches, tube diameter 0.9 mm, 30 mm long tube

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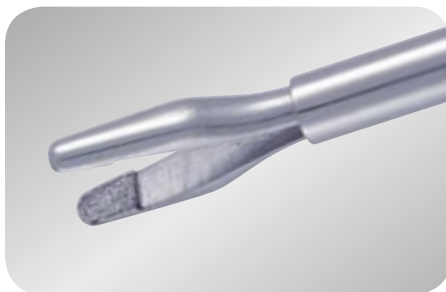Micro needle holder, 3.0 mm long branches, TC coated tips, tube diameter 1.2 mm, 22 mm long tube

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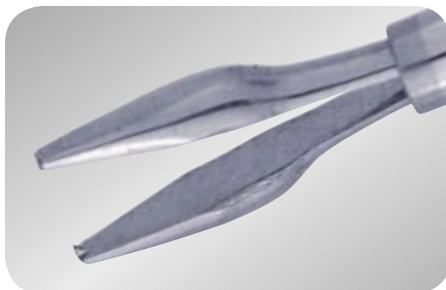
Forceps, side-grabbing, small, closed in normal position, 2.3 mm high branches, length of travel of movable part 3.0 mm, tube diameter 0.9 mm, 30 mm long tube

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Forceps, straight, with micro tooth, 1.8 mm long branches, tube diameter 0.9 mm, 30 mm long tube

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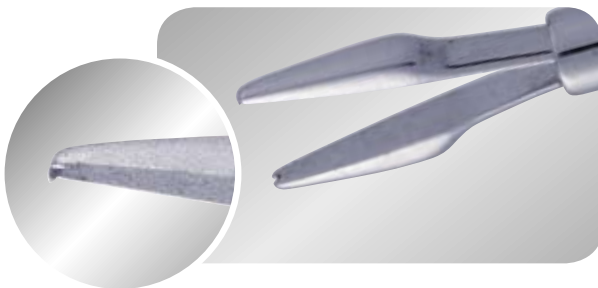
Forceps, straight, with micro tooth, 2.2 mm long branches, tube diameter 0.9 mm, 30 mm long tube

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Adaptor for washing the tips, connecting part to LUER syringes

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Subretinal spatula, narrowing working portion angled at 130°, 5 mm diameter round titanium handle, overall length 125 mm

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Membrane spatula, 0.65 mm wide working portion, angled at 110°, 5 mm diameter round titanium handle, overall length 125 mm

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Membrane spatula, 0.65 mm wide working portion, angled at 130°, 5 mm diameter round titanium handle, overall length 125 mm

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Membrane spatula, 0.65 mm wide working portion, angled at 150°, 5 mm diameter round titanium handle, overall length 125 mm

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Membrane spatula, 0.50 mm wide working portion, angled at 110°, 5 mm diameter round titanium handle, overall length 125 mm

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Membrane spatula, 0.50 mm wide working portion, angled at 130°, 5 mm diameter round titanium handle, overall length 125 mm

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Membrane spatula, 0.50 mm wide working portion, angled at 150°, 5 mm diameter round titanium handle, overall length 125 mm

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Membrane raker, 0.90 mm wide working portion, 5 mm diameter round titanium handle, overall length 125 mm

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Membrane spatula, drop-shaped working portion, 5 mm diameter round titanium handle, overall length 125 mm

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Membrane hook, with 0.45 mm diameter ball, 5 mm diameter round titanium handle, overall length 125 mm

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Membrane spatula, rounded, 0.90 mm wide and 2.5 mm long angled working portion, 5 mm diameter round titanium handle, overall length 125 mm

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Membrane spatula, rounded, 0.90 mm wide and 5.0 mm long angled working portion, 5 mm diameter round titanium handle, overall length 125 mm

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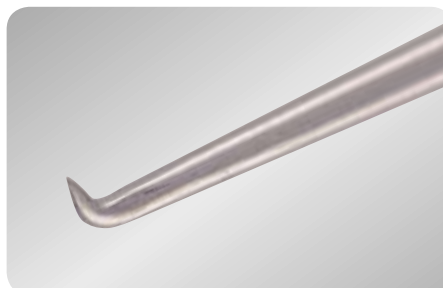
Membrane spatula, sharp-pointed, 0.90 mm wide and 5.0 mm long angled working portion, 5 mm diameter round titanium handle, overall length 125 mm

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
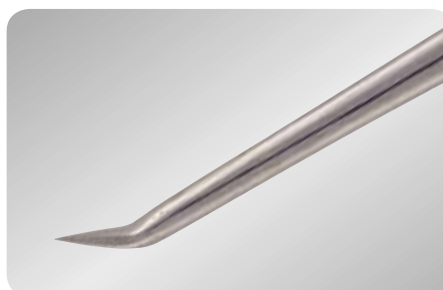
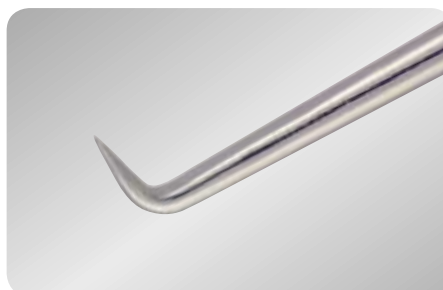
Membrane miniature pick, 1.0 mm long angled at 90°, 5 mm diameter round titanium handle, overall length 125 mm

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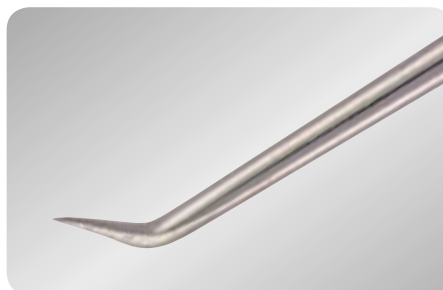Membrane miniature pick, 1.0 mm long angled at 120°, 5 mm diameter round titanium handle, overall length 125 mm

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Subretinal spatula, 4.0mm long 0.3 mm diameter round working portion angled at 120°, 5 mm diameter round titanium handle, overall length 125 mm

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Subretinal spatula, 6.0 mm long
0.3 mm diameter round working
portion angled at 120°, 5 mm
diameter round titanium handle,
overall length 125 mm

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Subretinal knife, 1.6 mm long
working portion with sharp inner
edge angled at 90°, 5 mm diameter
round titanium handle, overall
length 125 mm

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Subretinal knife, 1.6 mm long
working portion with sharp inner
edge angled at 135°, 5 mm
diameter round titanium handle,
overall length 125 mm

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Subretinal knife, 3.0 mm long
working portion with sharp inner
edge angled at 135°, 5 mm
diameter round titanium handle,
overall length 125 mm

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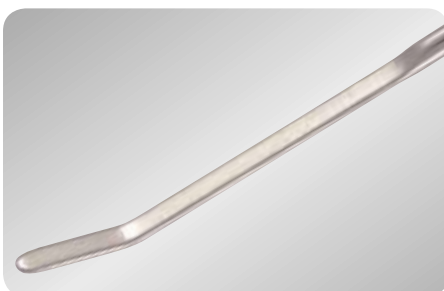
Membrane miniature pick, 0.5 mm
long, angled at 90°, 5 mm diameter
round titanium handle, overall
length 125 mm

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Membrane miniature spatula, 0.6
mm wide and 2.5 mm long working
portion angled at 150°, 5 mm
diameter round titanium handle,
overall length 125 mm

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Set of 23 G instruments
for seamless surgery (for multiple use)

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Trocar (instrument cannula), 23 G,
total length 5.7 mm,
stainless steel

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Cannula Plug,
total length 5.7 mm,
stainless steel

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Fixation ring, diameter of working
portion 7 mm, working portion
made of stainless steel, diameter
of titanium handle 5 mm,
total length 80 mm

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Cannula inserter, working portion
8 mm long, diameter 0.67 mm,
diameter of titanium handle 5 mm,
total length 58 mm

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Forceps for Cannula Removal,
working portion 2.1 mm x 2.8 mm,
total length 75 mm, titanium

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INSTRUCTION FOR OPERATION, CLEANING AND STERILIZATION

Microsurgical instruments consist of the tip and the universal handle. The set of delivery includes:

- | | |
|--|---------|
| 1. VN-008 universal handle | 1 unit |
| 2. The tip (microsurgical instrument) | 1 unit |
| 3. V-075 adaptor for cleaning the tip | 1 unit |
| 4. V-0131 protective case
(at customer's request) | 1 unit |
| 5. Instruction for Use | 1 copy. |

One handle may be included in the set consisting of various types and quantities of tips.

ASSEMBLY

In order to assemble the instrument one should screw the tip onto the handle. Check the assembled instrument by pressing the branches of the handle: the working portions of the instrument must perform reciprocating/alternating motions and the tip must rotate in respect to the handle. The instrument does not require any additional adjustments and is ready for operation.

CLEANING

Wash every instrument with distilled warm (30°C to 40°C) water immediately after the surgical operation. The inner part of the tips is washed with the help of V-075 adaptor. For that purpose disconnect the working tip from the handle, screw the adaptor onto the tip, attach a 50 ml to 100 ml syringe filled with water to the adaptor and push the water through the tip. Wash the tip with alcohol to remove the moisture from the tip. Dry the tip through the adaptor with compressed air to remove completely the alcohol from it.

Dry the handle and the tip at the temperature of the air plus 25°C to 50°C.

STERILIZATION

Sterilize the instruments in an autoclave or using dry-heat treatment procedure.

STORING

Keep the microsurgical instruments in the manufacturer's package or the sterilization case in closed premises at the temperature from + 5°C до + 40°C and the relative humidity of the air 80 % at the temperature + 25°C. The air inside the premises should not contain any corrosion-active admixtures.

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
GUARANTEE OBLIGATIONS

The guarantee period of use is 1 (one) year from the day of starting the use but not later than 18 months from the date of fabrication.

INFORMATION ON CLAIMS

Any claim together with the label should be submitted in the due course and sent to Manufacturer «Microsurgical Instruments Co., Ltd.» at the following address:
12, Salikha Saidasheva street, 420021, Kazan, Russia.
Tel. +7-843-278-28-05, +7-843-293-33-23;
e-mail: micro@kzn.ru ; micro@bancorp.ru
www.micro-instruments.com

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